Explanation of Symbols Which Appear on the Device Labeling
 Explicación de símbolos que aparecen en la etiqueta del dispositivo
 Erklärung der Symbole auf dem Etikett dieses Gerätes
 Explication des symboles qui apparaissent sur l'étiquette de l'appareil
 Spiegazione dei simboli che figurano nelle etichette del dispositivo
 Toelichting bij symbolen die op de etiketten van het apparaat staan
 Laitteen etiketissä olevien symbolien selitykset
 装置标签上的符号说明
 装置のラベルにある記号の意味
 Объяснение на символите върху етикетите на устройството
 Objašnjenje simbola koji se javljaju na etiketi uređaja
 Vysvětlení symbolů uvedených na štítku přístroje
 Επεξήγηση των συμβόλων που εμφανίζονται στην ετικιτάση της συσκευής
 Az eszköz címkézésén szereplő szimbólumok magyarázata
 Explicatia Simbolurilor Care Apar pe Eticheta Dispozitivului
 Vysvetlivky k symbolom, ktoré sa objavujú na výrobných štítkoch zariadení

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MiniMe® 2 nazalna ventilaciona maska

Nosna ventilacijska maska MiniMe® 2

These instructions contain the following languages:

English, Spanish, German, French, Italian, Dutch, Finnish, Chinese, Japanese, Bulgarian, Croatian, Czech, Greek, Hungarian, Romanian, Slovakian, Turkish, Portuguese, Swedish, Norwegian, Arabic,

Danish, Estonian, Latvian, Lithuanian, Polish, Russian, Serbian, and Slovenian

Sleepnet Corporation

5 Merrill Industrial Drive

Hampton, NH 03842 USA

Tel: (603) 758-6600

Fax: (603) 758-6699

Toll Free: 1-800-742-3646 (USA ONLY)

www.sleepnetmasks.com

	Caution or Warning Precaución o advertencia Achtung oder Warnung Avertissement ou mise en garde Attenzione o avvertenza Attentisignaal of waarschuwing Huomautus tai varoitus 注意事項或警告 注意・警告 Внимание или Предупреждение	Opzet ili Upozoreenje Upozornění nebo varování Προσοχή ή Προειδοποίηση Figyelmeztetés vagy óvintézkedés Atenționare sau avertisment Upozornenie alebo varovanie Ikaz veya Uyarı Precauções ou advertências Forsiktighetsåtgärd eller varning	Varsel eller advarsel تنبيه أو تحذير Forsigtig eller Advarsel Ettevaatus või hoiatus Uzmanību vai Brīdinājums Perspējamas arba iespējamas Przeszroga lub ostrzeżenie Внимание или Осторожно Mera opreza ili upozorenje Svarilo ali opozorilo
--	--	--	---

	Manufacturer Fabricante Hersteller Fabricant Fabbricante Fabrikant Valmistaja 製造商 製造元 Производитель Proizvodac Výrobce	Κατασκευαστής Gyártó Producător Výrobca Üretici Fabricante Tillverkare Fabrikant الجهة المصنعة	Gamintojas Producuent Изготовитель Proizvođač Izdelovalec
--	---	--	---

	Part number Número de pieza Teilenummer Référence Número di articolo Onderdeelnummer Osanumero 部件事号 部品番号 Номер на частта Broj dijela Číslo dílu	Αριθμός εξαρτήματος Cikkszám Număr componentă Číslo dielu Parça numarası Número de peça Art.nr Delenummer رقم القطعة	Dalies numeris Numer katalogowy Номер по каталогу Serijski broj Številka dela
--	--	--	---

	This Product is not manufactured with Natural Rubber Latex. Este producto no se fabrica con látex de caucho. Dieses Produkt ist nicht mit Kautschuklatex hergestellt Ce produit est fabriqué sans latex Il prodotto non contiene lattice. Bevat geen latex van natuurlijk rubber Tuoteen valmistuksessa ei ole käytetty luonnon kumilateksia. 本产品不含天然乳胶。 この製品には天然ゴムラテックスは使用されていません。 Този продукт не е произведен от естествен гуген латекс	Ovaj proizvod nije proizveden koristeći lateks od prirodne gume. Tento výrobek není vyroben z přírodního latexu. Αυτό το προϊόν δεν έχει κατασκευαστεί με φυσικό καουτσούκ. Ez a termék természetes gumi latex felhasználása nélkül készült Acest produs nu este fabricat cu cauciuc natural din latex. Tento výrobek nie je vyrobený z prírodného latexu. Bu Ürün Doğal Kauçuk Lateksten üretilmemiştir. Este produto não é fabricado com látex de borracha natural	Denna produkt har inte tillverkats med naturgummilátex. Dette produktet er ikke produsert med naturligt gummilátex. هذا المنتج غير مُصنَع باستخدام اللاتكس الطبيعي. Produktet indeholder ikke naturgummi/ latex. Acest produs nu este fabricat cu cauciuc natural din latex. Šis izstrādājums ir izgatavots, neizmantojot dabīgo gumijas lateksu.	Gaminant šį gaminį, nebuvo naudojamas natūralus kaučiuko lateksas. Produkt nie zawiera lateksu naturalnego. Настоящий продукт изготовляется без применения натурального каучукового латекса. Ovaj proizvod nije proizveden korišćenjem prirodne gume – lateksa. Ta izdelke ni izdelan iz naravnega lateksa.
--	---	---	--	---


	Date of Manufacture Fecha de fabricación Herstellungsdatum Date de fabrication Data di produzione Vervaardigingsdatum Valmistuspäivämäärä 製造日期 製造年月日 Дата на производство Datum proizvodnje Datum výroby Ημερομηνία	κατασκευής A gyártás időpontja Data producerii Datum výroby Üretim Tarihi Data de fabrico Tillverkningsdatum Fabrikasjonsdato تاريخ التصنيع	Data produkciji Дата изготовления Datum proizvodnje Datum izdelave
--	---	--	---

	Not MRI Safe No segura para resonancia magnética Nicht MRT/MRI-geschützt Ne pas être utilisé pendant une IRM Non compatible con la risonanza magnetica Niet bestand tegen MRI Ei MRI-turvallinen MRI 不安全 MRIに対して安全ではありません He e beaonoano pri RMP Nije sigurno za uporabu s uredajima za magnetsku rezonancijsku (MRI)	Eni bezpečné pro vyšetření magnetickou rezonancí Οχι ασφαλής σε μαγνητικό αλληλεπίδραση Nem MRI-biztos Fără siguranță IRM Nie je bezpečná pre magnetickú rezonanciu MRI için Uygun Değil Não seguro para ressonância magnética Får ej anvendes vid magnetrøntgen Ikke MRI-sikker غير آمن حين الاستخدام مع التصوير بالرنين المغناطيسي	Ei ole MRT-ohutu Produktis nav magnētiskās rezonances tomogrāfijas drošs Netinkama naudoti magnetinio rezonanso įrenginyje Nie nadaje się do bezpiecznego stosowania w srodowisku rezonansu magnetycznego He beaonoano las MPT Nije bezbedno za MRT Ni vamo za uporabo z MRI
--	---	---	--


Australian Sponsor:
 Breathing and Medical.
 127 Bowden St, Meadowbank, NSW 2114
 www.breathingandmedical.com.au
 1800 335 333

CE 0123
 90119 Rev. H

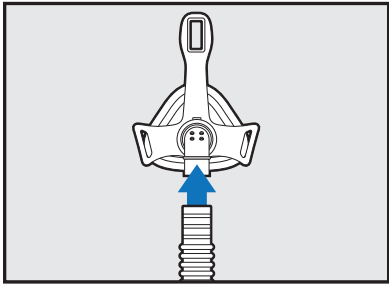
1


2

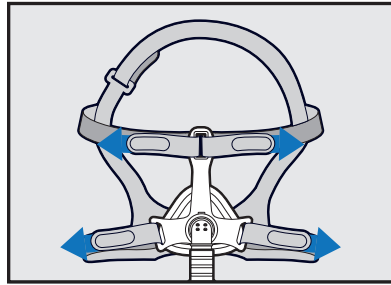3

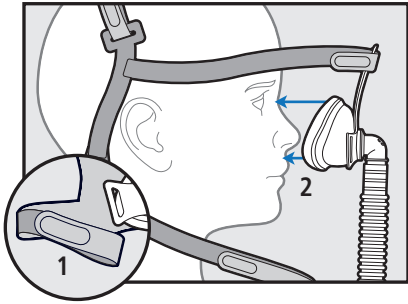
4

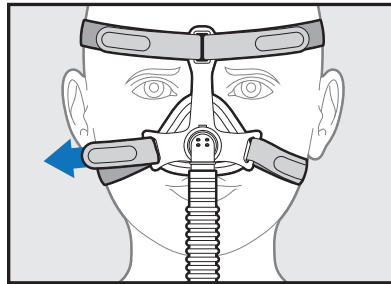
This page intentionally left blank.

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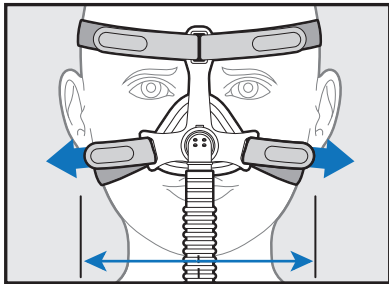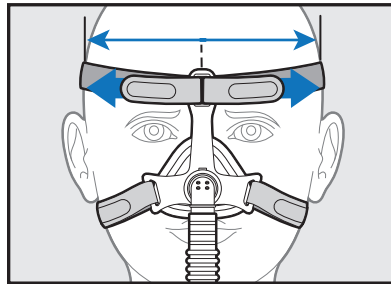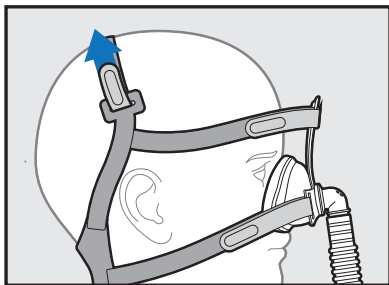
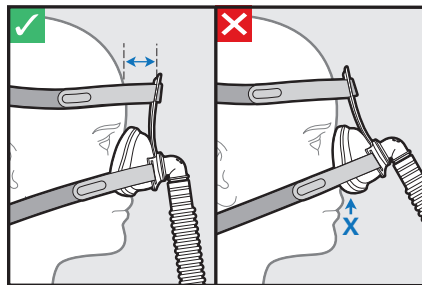
MiniMe® 2 Nasal Vented Mask

CONTENTS: MiniMe® 2 Nasal Vented Mask, tubing, and headgear
THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX
Rx ONLY MADE IN U.S.A.

The MiniMe 2 Pediatric Nasal Vented Mask is intended to provide a patient interface for application of positive pressure therapy. The mask is to be used as an accessory to CPAP or Bi-level positive pressure systems. It is intended for patients >2 years old and <12 years old. It is intended for single patient, multi-use in the home, hospital, or institutional environment.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the mask to temperatures above 122°F (50°C).

WARNING: Do not use the Mask with baseline pressure less than 4 cm H₂O or greater than 20 cm H₂O.

CAUTION: The mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: This mask is not suitable for providing life support ventilation.

WARNING: Do not block or try to seal the vent holes.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. The mask is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

WARNING: Using a mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

WARNING: Significant unintentional leak may occur without properly fitting the mask. Follow the USER INSTRUCTIONS FOR MINIME 2 NASAL MASK provided.

WARNING: If the mask is prescribed for children with serious congenital conditions or cognitive impairments then the positive pressure device must have the appropriate alarms. The device, mask, and alarms must be tested before starting therapy.

GENERAL INFORMATION

The mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit <http://www.sleepnetmasks.com/products/our-promise.cfm>.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Hand wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.90 cmH₂O

Resistance to air flow at 100 L/min: 3.60 cmH₂O

Dead space (approx): **XS size-** 20 ml **S size-** 25 ml **M size-** 37 ml **L size-** 40 ml

A-weighted Sound Pressure Level at 1 m distance: 29.89 dBA; A-weighted Sound Power Level: 32.93 dBA

Operating Temperature: Do not expose the mask to temperatures above 122°F (50°C)

USER INSTRUCTIONS FOR THE MINIME® 2 NASAL MASK

1. Attach the tubing assembly to the MiniMe 2 Mask (Figure 1).
2. Lay headgear flat with light gray side facing up with the head strap at the top as shown in Figure 2. Place mask gel side down in the center of the headgear and thread all 4 straps through the slots as shown (Figure 2).
3. To place mask on child, detach one or both lower straps from the mask. Slip the headgear over the head. Seat the bottom cushion of the mask between the upper lip and nose, and tip the mask up to contact the nose (Figure 3).

NOTE: The Touchless Spacebar™ should not contact the forehead.

4. Find the detached hook and loop fastener strap and reattach (Figure 4).
5. Ensure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head.
6. Adjust the bottom headgear straps by using the hook and loop fastener tabs (Figure 5).
7. Adjust the upper headgear straps by using the hook and loop fastener tabs: detach the hook and loop fastener tabs then pull straps forward to center the mask over face. Ensure headgear is centered by aligning the two straps. Then pull straps back, adjust strap tension, and affix tabs (Figure 6).
8. Adjust the head strap by using the hook and loop fastener tab (Figure 7).
9. If mask is uncomfortable repeat steps 6-9.

NOTE: Do NOT overtighten the headgear.

NOTE: The Touchless Spacebar™ should NOT touch the forehead. It should be parallel to the forehead (Figure 8).

NOTE: For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the straps, tightening or loosening the straps, and reattaching the hook and loop fastener tabs.

10. Connect the mask tubing assembly to the positive airway pressure device tubing and turn the device on to the prescribed pressure (Figure 9).
11. Gently press the edges of the mask to change mask shape, create a comfortable fit, and reduce leaks.
NOTE: The MiniMe 2 Nasal Mask contains Custom Fit Technology™. This soft mask shell can be shaped to follow the contours of a child's face. The Memory Wire™ embedded into the shell locks the customized fit in place.
12. Turn head from side to side while lying on the bed and identify areas of leak that may prevent sleep.
13. If leaks are noticed at the bridge of the nose, slightly tighten the top straps as described in in step 7. The Touchless Spacebar™ should not contact the forehead.
14. If leaks are noticed at the upper lip or below the nose, slightly tighten the bottom straps.

MiniMe® 2 is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com.

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A MiniMe® 2 a Sleepnet Corporation bejegyzett védjegye.