Explanation of Symbols Which Appear on the Device Labeling
Explicación de símbolos que aparecen en la etiqueta del dispositivo
Erklärung der Symbole auf dem Etikett dieses Gerätes
Explication des symboles qui apparaissent sur l'étiquette de l'appareil
Spiegazione dei simboli che figurano nelle etichette del dispositivo
Toelichting bij symbolen die op de etiketten van het apparaat staan
Laitteen etiketissä olevien symbolien selitykset
装置标签上的符号说明
装置のラベルにある記号の意味
Объяснение на символите върху етикетите на устройството
Objasnenje simbola koji se javljaju na etiketi uređaja
Vysvětlení symbolů uvedených na štítku přístroje
Επεξηγήση των συμβόλων που εμφανίζονται στην επισήμανση της συσκευής
Az eszköz címkézésén szereplő szimbólumok magyarázata
Explicação Simbolorilor Care Apar pe Eticheta Dispozitivului

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MiniMe® 2 nazalna neventilirana maska
Nosna maska brez ventilacije MiniMe® 2

These instructions contain the following languages:

English, Spanish, German, French, Italian, Japanese, Dutch, Finnish, Chinese, Bulgarian, Croatian, Czech, Greek, Hungarian, Romanian, Slovakian, Turkish, Portuguese, Swedish, Norwegian, Arabic, Danish, Estonian, Latvian, Lithuanian, Polish, Russian, Serbian, and Slovenian

Sleepnet Corporation
5 Merrill Industrial Drive
Hampton, NH 03842 USA
Tel: (603) 758-6600
Fax: (603) 758-6699
Toll Free: 1-800-742-3646 (USA ONLY)
www.sleepnetmasks.com

Caution or Warning Precaución o advertencia Achtung oder Warnung Avertissement ou mise en garde Attenzione o avvertenza Auteentingsaal of waarschuwing Hoortuust tai varoitus 注意事項或警告 注意・警告 Внимание или Предупреждение Opzetsj Urozorenje	Upozornění nebo varování Προσοχή ή Προειδοποίηση Figyelemzésés vagy óvintézkedés Attentionné sau Avertisment Upozornenie alebo varovanie Ikaz veya Uyarı Precauções ou advertências Förstjrhetsåtgård eller varning Varsel eller advarsel تنبیه أو تحذیر	Forsigtig eller Advarsel Ettevaatus või hoiatus Uzmaniibu vai Brjndinjums Perspejimas arba įspėjimas Przestroga lub ostrzezenie Внимание или Осторожно Meta opeza ili upozorenje Svarilo ali opozorilo
--	---	---

CE Certification Mark Marca de certificación CE Zertifizierungsmarke Marque CE Marcatura CE EG-keurmerk CE-certifiointmerkintä CE 认证标记 CE маркировка CE oznaka Označeni CE certifikace	Σήμα πιστοποίησης CE CE tanusitivány-jelölés Marcacj certificare CE Registračná známka CE CE Belgelendirmne [careti Marca de certificaçáo CE CE-mårkning CE-certifiseringmerke علامة المطابقة للمعايير الأوروبية CE certifikace CE-certificeringsmärke	CE-certifitseerimismårk CE sertifikata zime CE sertifikavimo ženklas Znak CE Знак сертификации CE CE oznaka Oznaka CE
---	--	---

Lot number Número de lote Losnummer Número du lot Numero di lotto Partijnummer Erlänummer CE-oznaka Broj serije	Číslo šarže Αριθμός παρτίδας Gyártási tétel száma Numár lot Číslo šarže Parti numaras CE oznaka Partijnummer Partinummer Partinummer رقم التغليف Partinummer	Partii number Partijas numurs Partijos numeris Numer partii Номер партии Broj šarže Številka serije
---	---	---

Part number Número de pieza Teilenummer Référérence Numero di articolo Onderdeelnummer Osnummer Art.nr Delenummer رقم القطعة Delnummer	Číslo dílu Αριθμός εξαρτήματος Cikkszám Numár componentă Číslo dielu Parça numarası Número de peça Art.nr Delenummer رقم القطعة Delnummer	Osa number Dalas numurs Dalties numeris Numer katalogowy Номер по каталогу Šerijski broj Stevilka dela
--	---	--

Date of Manufacture Fecha de fabricación Herstellungsdatum Date de fabrication Data di produzione Vervaardigingsdatum Valmistuspäivämäärä 製造年月日 [ata na proizvodstvo Datum proizvodnje	Datum výroby Ημερομηνία κατασκευής A gyártás időpontja Data producerii Datum výroby Üretim Tarihi Data de fabrico Tiliverkningsdatum Fabrikasjonsdato تاريخ التصنيع	Fremstillingsdato Tootmiskupäev Izgatavošanas datums Pagaminimo data Data produkci Дата изготовления Datum proizvodnje Datum izdelave
---	--	--

Not MRI Safe No segura para resonancia magnetica Nicht MRT/MRI-geschützt Ne pas être utilisé pendant une IRM Non compatible con la risonanza magnetica Niet bestand tegen MRI Ei MRI-vaikallinen MRI 不安全 MRIに対して安全ではありません He e beoapno pri RMP Nije sigurno za uporabu s uredajima za magnetsku rezonancijsku (MRI)	Neni bezpeční pro vyšetření magnetickou rezonancí Οχι ασφαλής σε περιβάλλον μαγνητικού τοιούρουρου Nem MRI-biztos Fără siguranță IRM Nie je bezpečná pre magnetickú rezonanciu MRI için Uygun Değil Náo seguro para ressonância magnetica Får ej användas vid magnetröntgen Ikke MRI-sikker غير آمن حين الاستخدام مع التصوير بالرنين المغناطيسي Ikke MR-sikker	Ei ole MRT-ohutu Produkts nav magnētiskās rezonanses tomogrāfijas nodrošinātā Netinkama naudoti magnetinio rezonanso įengimyje Nie nadaje się do bezpiecznego stosowania w środowisku rezonansu magnetycznego He beoapno dla MPT Nije bezbedno za MR Ni varno za uporabo z MRI
---	--	--

This Product is not manufactured with Natural Rubber Latex. Este producto no se fabrica con látex de caucho. Dieses Produkt ist nicht mit Kautschuklatex hergestellt Ce produit est fabriqué sans latex Il prodotto non contiene lattice. Bevat geen latex van natuurlijk rubber Tuotten valmistuksessa ei ole käytetty luonnonkumilätekсия 本产品不含天然乳胶 この製品には天然ゴムラテックスは使用されていません。	Този продукт не е произведен от естествен гумеи латекс Ovaj proizvod nije proizveden koristeći lateks od prirodne gume. Тento výrobek neni vyroben z prírodného latexu. Αυτό το προϊόν δεν έχει κατασκευαστεί με φυσικό κωουτσόφιν. Ez a termék termézetes gumi latex felhasználása nélkül készült Acest produs nu este fabricat cu cauciuc natural din latex. Tento výrobek nije je vyrobený z prírodného latexu.	Bu Ürün Doğal Kauçuk Lateks ten üretilmemiştir. Este produto não é fabricado com látex de borracha natural Denna produkt har inte tillverkats med naturgummlatex. Dette produktet er ikke produsert med naturlig gummlatex. Produktet indeholder ikke naturgummi/ latex.	Toote valmistamisel ei ole kasutatud looduslikku kummlatexit Šis izstrādājums ir izgatavots, neizmantojot dabīgo gumijas lateksu. Gaminant ši gamini, nebuvo naudojamasi natūralus kaučiuko lateksas. Produkt nie zawiera lateksu naturalnego Настоящий продукт изготавливается без применения натурального каучукового латекса. Ovaj proizvod nije proizveden koristećenjem prirodne gume – lateksa. Ta izdelek ni izdelan iz naravnega lateksa.
--	--	--	---


Australian Sponsor:
Breathing and Medical.
127 Bowden St, Meadowbank, NSW 2114
www.breathingandmedical.com.au
1800 335 333


910136 Rev. H

EU Rep:
Medical Product Services GmbH
Borngasse 20, 35619 Braunfels Germany
info@mps-gmbh.eu


1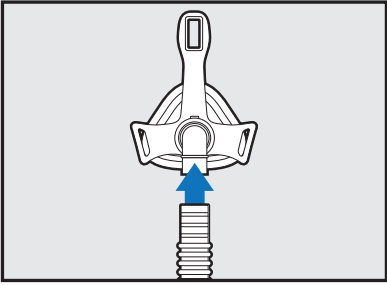


2

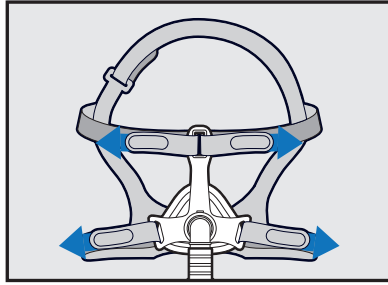3

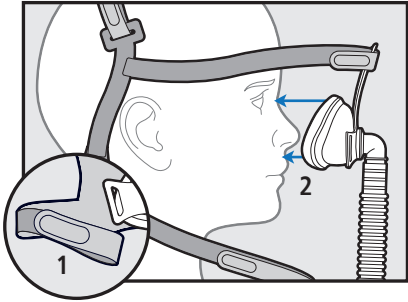
4

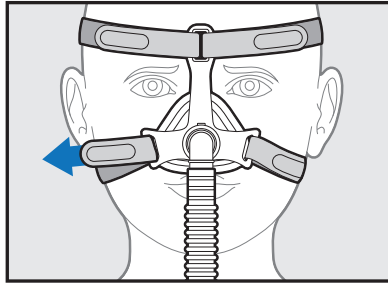
This page intentionally left blank.


5

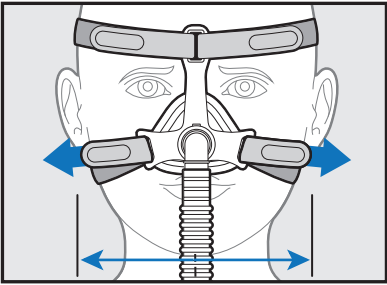
6


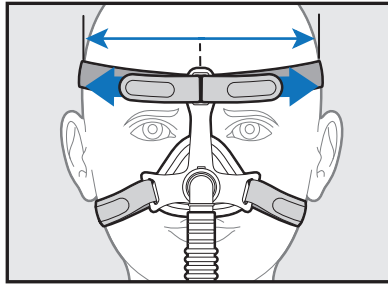
7

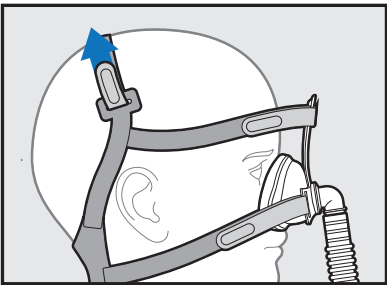8

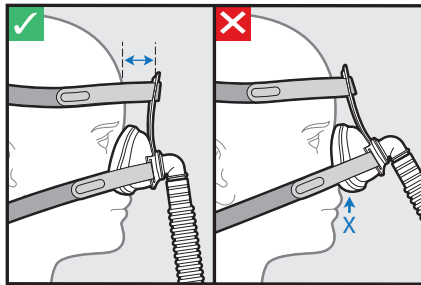This page intentionally left blank.

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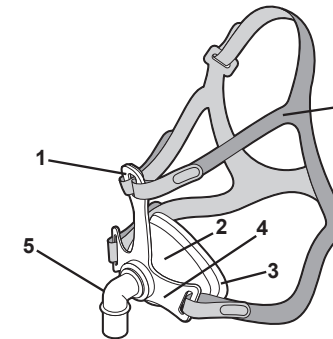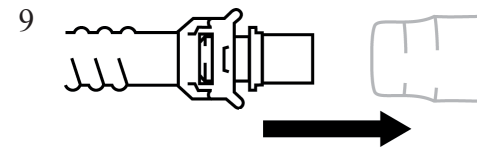MiniMe® 2 Nasal Non-Vented Mask

CONTENTS: MiniMe® 2 Pediatric Nasal Non-Vented Mask, Tubing, and Headgear
THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX
Rx ONLY MADE IN U.S.A.

The MiniMe 2 Pediatric Nasal Non-Vented Mask is intended to provide an interface for application of positive pressure therapy. The mask is to be used as an accessory to CPAP or Bi-level positive pressure systems CONTAINING EXHALATION VALVES. It is intended for patients >2 years old and <12 years old. It is intended for single patient, multi-use in the home, hospital, or institutional environment.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the mask to temperatures above 122°F (50°C).

WARNING: Do not use the Mask with baseline pressure less than 4 cm H₂O or greater than 20 cm H₂O.

CAUTION: The mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: This mask is not suitable for providing life support ventilation.

WARNING: The mask does not have exhalation holes and must be used with a patient circuit that incorporates an exhalation valve.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. The mask is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The vent holes in the patient circuit should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, air from the device flushes the exhaled air out through the vent holes in the patient circuit. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

WARNING: Using a mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

WARNING: Significant unintentional leak may occur without properly fitting the mask. Follow the USER INSTRUCTIONS FOR MINIME 2 NASAL MASK provided.

WARNING: If the mask is prescribed for children with serious congenital conditions or cognitive impairments then the positive pressure device must have the appropriate alarms. The device, mask, and alarms must be tested before starting therapy.

GENERAL INFORMATION

The mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit <http://www.sleepnetmasks.com/products/our-promise.cfm>.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX

NOTE: The mask without vent holes can only be used with positive pressure devices that have an exhaust valve.

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Upor pri pretoku zraka 50 l/min: 1,10 cmH₂O

Upor pri pretoku zraka 100 l/min: 4,20 cmH₂O

Mrtvi prostor (pribl.): velikost **XS** – 20 ml, velikost **S** – 25 ml, velikost **M** – 37 ml, velikost **L** – 40 ml, cevje – 35 ml

Delovna temperatura: Maske ne izpostavljajte temperaturam, višjim od 50 °C (122 °F).

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CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Hand wash the mask and components in warm water using a mild detergent. Rinse thoroughly after washing. Visually verify that the swivel elbow is not obstructed and allow the mask to air dry.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 1.10 cmH₂O

Resistance to air flow at 100 L/min: 4.20 cmH₂O

Dead space (approx): **XS size**- 20 ml, **S size**- 25 ml, **M size**- 37 ml, **L size**- 40 ml, **Tubing**- 35 ml

Operating Temperature: Do not expose the mask to temperatures above 122°F (50°C)

USER INSTRUCTIONS FOR THE MINIME 2 NASAL MASK

1. Attach the tubing assembly to the MiniMe 2 Mask (Figure 1).
2. Lay headgear flat with light gray side facing up with the head strap at the top as shown in Figure 2. Place mask gel side down in the center of the headgear and thread all 4 straps through the slots as shown (Figure 2).
3. To place mask on child, detach one or both lower straps from the mask. Slip the headgear over the head. Seat the bottom cushion of the mask between the upper lip and nose, and tip the mask up to contact the nose (Figure 3).
NOTE: The Touchless Spacebar™ should not contact the forehead.
4. Find the detached hook and loop fastener strap and reattach (Figure 4).
5. Ensure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head.
6. Adjust the bottom headgear straps by using the hook and loop fastener tabs (Figure 5).
7. Adjust the upper headgear straps by using the hook and loop fastener tabs: detach the hook and loop fastener tabs then pull straps forward to center the mask over face. Ensure headgear is centered by aligning the two straps. Then pull straps back, adjust strap tension, and affix tabs (Figure 6).
8. Adjust the head strap by using the hook and loop fastener tab (Figure 7).
9. If mask is uncomfortable repeat steps 6-9.
NOTE: Do NOT overtighten the headgear.
NOTE: The Touchless Spacebar™ should NOT touch the forehead. It should be parallel to the forehead (Figure 8).
NOTE: For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the straps, tightening or loosening the straps, and reattaching the hook and loop fastener tabs.
10. Connect the mask tubing assembly to the positive airway pressure device tubing and turn the device on to the prescribed pressure (Figure 9).
11. Gently press the edges of the mask to change mask shape, create a comfortable fit, and reduce leaks.
NOTE: The MiniMe 2 Nasal Mask contains Custom Fit Technology™. This soft mask shell can be shaped to follow the contours of a child's face. The Memory Wire™ embedded into the shell locks the customized fit in place.
12. Turn head from side to side while lying on the bed and identify areas of leak that may prevent sleep.
13. If leaks are noticed at the bridge of the nose, slightly tighten the top straps as described in in step 7. The Touchless Spacebar™ should not contact the forehead.
14. If leaks are noticed at the upper lip or below the nose, slightly tighten the bottom straps

 **MiniMe® 2** is a trademark of Sleepnet Corporation.
Visit our website at www.sleepnetmasks.com.

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