Explanation of Symbols
Explicación de símbolos
Erklärung der Symbole
Explication des symboles
Toelichting bij symbolen
Spiegazione dei simboli
Επεξήγηση των συμβόλων

Forklaring på symboler
Förklaring av symboler
Explicação dos símbolos
Wyjaśnienie symboli
Merkkien selitys
Sembollerin açıklaması
Symbolforklaring

Caution or Warning
 Precaución o advertencia
 Achtung oder Warnung
 Avertissement ou mise en garde
 Attentiesignaal of waarschuwing
 Attenzione o avvertenza
 Επισημάνση προσοχής ή προειδοποίηση
 Forsigtighed eller advarsel
 Försiktighet eller varning
 Cuidado ou advertência
 Uwaga lub ostrzeżenie
 Huomautus tai varoitus
 Υπάρει ve Önlem
 Varsel eller advarsel

Lot number
 Número de lote
 Losnummer
 Numéro du lot
 Partijnummer
 Numero di lotto
 Αριθμός παρτίδας
 Varenummer
 Partinummer
 Número do lote
 Numer partii
 Eränumero
 Lot numarası
 Partinummer

Manufacturer
 Fabricante
 Hersteller
 Fabricant
 Fabrikant
 Fabbricante
 Κατασκευαστής
 Producent
 Tillverkare
 Fabricante
 Producent
 Valmistaja
 Üretici
 Fabrikant

Does not contain natural rubber latex
 No contiene látex de caucho natural
 Enthält keinen Naturkautschuklatex
 Ne contient pas de latex en caoutchouc naturel
 Bevat geen latex van natuurlijk rubber
 Non contiene lattice di gomma naturale
 Δεν περιέχει λάτεξ από φυσικό καουτσούκ
 Inneholder ikke naturgummilatex
 Innehåller inte rågummilatex
 Não contém látex de borracha natural
 Nie zawiera lateksu naturalnego.
 Ei sisällää luonnonkumista valmistettua lateksia.
 Doğal kauçuk lateks içermez.
 Inneholder ikke naturlig gummilatex

Part number
 Número de pieza
 Teilenummer
 Référence
 Onderdeelnummer
 Numero di articolo
 Αριθμός εξαρτήματος
 Del nummer
 Delnummer
 Número da peça
 Numer części
 Osanumero
 Parça numarası
 Delenummer

Medical Device
 Producto sanitario
 Medizinisches Gerät
 Dispositif médical
 Medisch hulpmiddel
 Dispositivo medico
 Ιατρική Συσκευή
 Medicinsk utrustning
 Medicinsk enhed
 Dispositivo médico
 Urządzenie medyczne
 Lääkinnällinen laite
 Medikal Cihaz
 Medisinsk enhet

Not MRI Safe
 No segura para resonancia magnética
 Nicht MRT/MRI-geschützt
 Ne pas être utilisé pendant une IRM
 Non compatibile con la risonanza magnetica
 Niet bestand tegen MRI
 Non compatibile con la risonanza magnetica
 Όχι ασφαλές σε περιβάλλον μαγνητικού τομογράφου
 Ikke MR-sikker
 Får ej användas vid magnetröntgen
 Não seguro para ressonância magnética
 Nie nadaje się do bezpiecznego stosowania w środowisku rezonansu magnetycznego
 Ei MRI-turvallinen
 MRI için Uygun Değil
 Ikke MRI-sikker

Use by Date
 Fecha de caducidad
 Verfallsdatum
 Date limite d'utilisation
 Uiterste houdbaarheidsdatum
 Da utilizzare entro la data indicata
 Χρησιμοποιήστε μέχρι την Ημερομηνία
 Bör användas före
 Holdbarhedsdato
 Prazo de validade
 Data przydatności do użycia
 Viimeinen käyttöpäivä
 Son Kullanma Tarihi
 Brukes innen

Importer
 Importador
 Importateur
 Importateur
 Importeur
 Importatore
 Εισαγωγέας
 Importör
 Importör
 Importador
 Importer
 Maahantuojaja
 İthalatçı
 Importer

Single use only
 Un solo uso
 Nur für einzigen Patienten verwenden
 Usage unique seulement
 Voor eenmalig gebruik
 Exclusivamente monouso
 Για μία μόνο χρήση
 Kun enkelt brug
 Endast för engångsbruk
 Apenas para utilização única
 Tylko do jednorazowego użytku
 Vain yhden potilaan käyttöön
 Tek kullannik
 Kun enkeltbruk

Medical Product Services GmbH
 Borngasse 20, 35619 Braunfels Germany
 info@mps-gmbh.eu

User Instructions for MiniMe® Pediatric Ventilation Mask and Headgear

Instrucciones de usuario para la máscara pediátrica de ventilación y el casco MiniMe®
 Gebrauchsanleitung für MiniMe® Pädiatrische Beatmungsмасke mit Kopfgurt
 Mode d'emploi du casque du masque de ventilation pédiatrique MiniMe®
 Gebruiksaanwijzing voor het pediatische ventilatiemasker en hoofdhulsel MiniMe®
 Istruzioni per l'uso della cuffia e della Maschera pediatrica per ventilazione MiniMe®

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These instructions contain the following languages:
 English, Spanish, German, French, Dutch, Italian, Greek,
 Swedish, Danish, Portuguese, Polish, Finnish, Turkish, and
 Norwegian

Sleepnet Corporation

5 Merrill Industrial Drive

Hampton, NH 03842 USA

Tel: (603) 758-6600

Fax: (603) 758-6699


Toll Free: 1-800-742-3646 (USA ONLY)

www.sleepnetmasks.com


0123
 90137 Rev. E 2023-01

1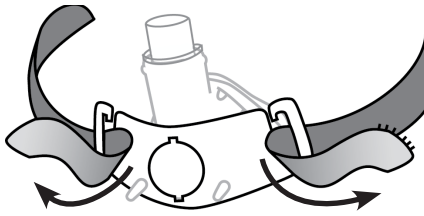

2

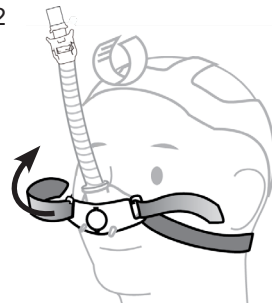3

4


Mask tubing
 Tubo de máscara
 Maskenschlauch
 Tubulure du masque
 Slang masker
 Tubo della maschera
 Σωληνάκι μάσκας
 Maskeslange
 Maskslang
 Tubo da máscara
 Rurka maski
 Maskin letkujärjestelmä
 Maske hortumu
 Maskeslanger

Ventilator tubing
 Tubo del respirador
 Ventilatorschlauch
 Tubulure de l'appareil de ventilation
 Ventilatorslang
 Tubo per la ventilazione
 Σωλήνας εξαερισμού
 Ventilatorslange
 Slang från andningsaggregat
 Tubo de ventilador
 Rurka aparatu oddechowego
 Hengityslaitteen letku letkujärjestelmä
 Ventilatör hortumu
 Ventilatorslanger

This page intentionally left blank.


Memory Wire™

5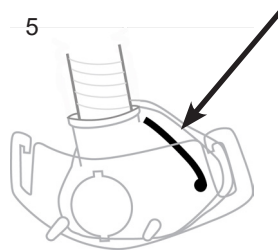


Squeeze
 Presionar
 Drücken
 Presser
 Premere
 Indrukken
 Πιέστε
 Tryk
 Tryck
 Apertar
 Ścisnąć
 Purista
 Sıkıştırma
 Klem


6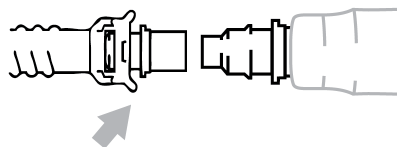

7

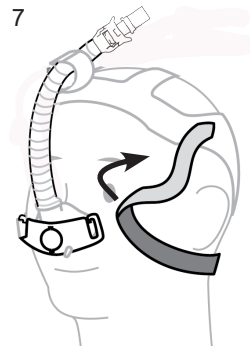This page intentionally left blank.

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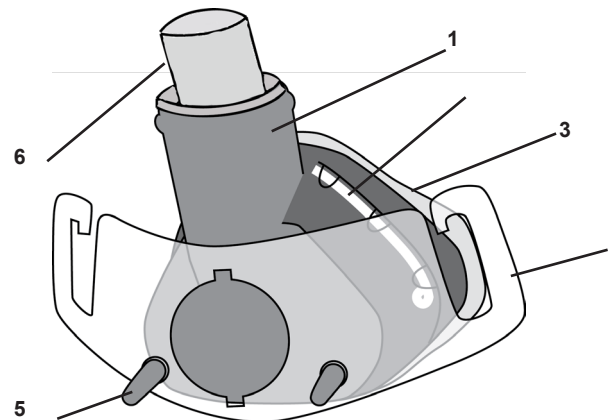MiniMe® Pediatric Ventilation Mask

ENGLISH

CONTENTS: MiniMe® Pediatric Ventilation Mask, Tubing Assembly, and Headgear

Rx ONLY MADE IN U.S.A.

DOES NOT CONTAIN NATURAL RUBBER LATEX

The MiniMe® Pediatric Ventilation Mask is intended to provide a patient interface for application of noninvasive ventilation. The mask is to be used as an accessory to ventilators that have adequate alarms and safety systems for ventilator failure, and which are intended to administer positive pressure ventilation for treatment of respiratory failure or respiratory insufficiency. It is intended for patients >2 years old and <12 years old. The MiniMe® Pediatric Ventilation Mask is intended for single patient, multi-use in the home and in the hospital/institutional environment.

CAUTIONS AND WARNINGS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the MiniMe® Pediatric Ventilation Mask to temperatures above 122°F (50°C).

WARNING: This mask should be used with POSITIVE PRESSURE VENTILATOR WITH AN

EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure ventilator is turned on and operating properly. When the ventilator is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the ventilator is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

WARNING: This mask must be used with ventilators which have ADEQUATE ALARMS AND SAFETY SYSTEMS in the event of a ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

CAUTION: The MiniMe® Pediatric Ventilation Mask is designed to provide a stable, effective air seal with minimum headgear tensio.

Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. Due to the size of the mask, it is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: If oxygen is used with this mask, the oxygen flow must be turned off when the CPAP device is not operating. Explanation of the warning: when the ventilator is not in operation, and the oxygen flow is left on, oxygen delivered into the tubing may accumulate within the ventilator enclosure. Oxygen accumulated in the ventilator enclosure will create a risk of fire.

WARNING: No smoking or open flames (such as candles) may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

GENERAL INFORMATION

The MiniMe® Pediatric Mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit <http://www.sleepnetmasks.com>. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: The MiniMe® Pediatric Mask and its accessories DO NOT CONTAIN NATURAL RUBBER LATEX.

NOTE: The MiniMe® Pediatric Mask is provided clean, non-sterile, and ready to use. No additional processing is required prior to the initial use.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new MiniMe® Pediatric Mask with care.

NOTE: The MiniMe® Pediatric Mask without vent holes can only be used with ventilators having an active exhaust valve.

CLEANING AND MAINTENANCE

For Home Use: Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild soap such as Ivory or Dove. Rinse the mask thoroughly after washing. Visually verify that there are no obstructions and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.38 cm H₂O; at 100 L/min: 1.51 cm H₂O

DEADSPACE VOLUME (approx.) with tubing 57 mL, without tubing 21 mL.

User Instructions for MiniMe® Pediatric Mask Headgear

1. Place headgear on head, with blue fabric facing outward. Hook both straps on the open slots in the headgear connector (Figure 1).
2. Pull straps outward to center the mask over nose. Adjust straps. Headgear sizes Small and Medium feature additional Velcro® tabs (Figure 2 and 3).

NOTE: The mask should comfortably cover the nose, and still provide an effective air seal. Straps should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask should not constrict the nose; child should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

3. Attach mask tubing to CPAP tubing; turn device on and verify airflow through mask (Figure 4).
4. If you notice any air leaks, gently press along the Memory Wire™ to seal the leaks (Figure 5).

NOTE: If leaks persist despite customization of Memory Wire and adjustment of headgear straps, try moving the mask higher or lower around the nose to find the best fit. Velcro headgear straps can be readjusted to improve fit as well.

REMOVING THE MASK AND HEADGEAR

5. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing (Figure 6).
6. Open one of the horizontal straps and detach from headgear connector on the mask (Figure 7).

MiniMe® is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com

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