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1. Øverste strop
2. Underste strop
3. Hovedtøjramme

Huvudset

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CONTENTS: **Mojo[®]** Non Vented Full Face Mask with Headgear
Rx ONLY MADE IN U.S.A.
DOES NOT CONTAIN NATURAL RUBBER LATEX

INTENDED USE

The **Mojo[®]** Non Vented Full Face Mask is intended to provide a patient interface for application of non-invasive ventilation. The mask is to be used as an accessory to ventilators that have adequate alarms and safety systems for ventilator failure, and which are intended to administer positive pressure ventilation. It is intended for use on adult patients (>30 kg.) who are appropriate candidates for noninvasive ventilation in the hospital or institutional environment.

CONTRAINDICATIONS

Do not use this mask if you or anyone (example: household members, bed partners, caregivers, etc.) in close physical contact with your mask has an active medical implant or metallic implant that will interact with magnets. Implant examples include, but are not limited to, pacemakers, implantable cardioverter defibrillators (ICD), neurostimulators, aneurysm clips, metallic stents, ocular implants, insulin/infusion pumps, cerebral spinal fluid (CSF) shunts, embolic coils, metallic splinter, implants to restore hearing or balance with implanted magnets (such as cochlear implants), flow disruption devices, contact lenses with metal, dental implants, metallic cranial plates, screws, burr hole covers, bone substitute device, magnetic metallic implants/electrodes/valves placed in upper limbs, torso, or higher, etc. If you have any questions regarding the implant, consult your physician or the manufacturer of your implant.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the **Mojo[®]** Non Vented Full Face Mask to temperatures above 122°F (50°C).

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: This mask is not intended to be used with a nebulizer.

WARNING: This mask is not suitable for providing life support ventilation.

WARNING: This mask should be used with a POSITIVE PRESSURE VENTILATOR WITH AN EXHALATION VALVE recommended by the manufacturer or by your physician or respiratory therapist. The mask should not be used unless the positive pressure ventilator is turned on and operating properly. When the ventilator is functioning properly, the exhalation valve allows exhaled air to escape to ambient. When the ventilator is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation.

WARNING: This mask must be used with ventilators which have adequate alarms and safety systems in the event of a ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: If using O₂ enrichment, turn off oxygen supply when not in use. When using a fixed flow rate of supplemental oxygen, the inhaled oxygen percentage will vary, depending on pressure settings, patient breathing pattern, mask size, and leak rate.

WARNING: The **Mojo[®]** Non Vented Full Face Mask must be used with non-conductive tubing.

WARNING: Do not use the **Mojo[®]** Non Vented Full Face Mask with baseline pressure less than 3 cm H₂O.

WARNING: To minimize the risk of vomiting during sleep, avoid eating or drinking for three (3) hours before using the mask.

WARNING: This product is indicated for use with POSITIVE PRESSURE VENTILATORS CONTAINING EXHALATION VALVES.

WARNING: This mask does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed. This mask should not be used for patients without a spontaneous respiratory drive. This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask. This mask may not be suitable for persons with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia.

CAUTION: Consult physician prior to using mask if using any medications or devices to remove deep secretions.

WARNING: Any unusual chest discomfort, shortness of breath, gastric distention, abdominal pain, belching or flatulence from ingested air, or severe headache during or immediately after use should be immediately reported to physician.

WARNING: Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult a physician or dentist.

WARNING: Consult a physician if you experience any of the following symptoms while using the mask: dry-
ing of the eyes, eye pain, eye infections, or blurred vision. Consult an ophthalmologist if symptoms persist.

CAUTION: Any unusual skin irritation should be immediately reported to physician.

Do not use this product if you are experiencing nausea, vomiting, taking a prescription drug that may
cause vomiting, or if you are unable to remove the mask on your own.

WARNING: Do not use in an MRI environment.

WARNING: Magnets are used in the mask and headgear clips with a field strength of 380mT. With the
exception of the devices identified in the contraindication, ensure that the mask is kept at least 6 inches
(approx.16 cm) away from any other medical implants or medical devices that can be impacted by the
magnetic fields to avoid possible effects from localized magnetic fields. For instance, the functioning of
implants may change, or implants may move within the body due to magnetic attraction/repulsion. This
applies to you or anyone in close physical contact with your mask.

GENERAL INFORMATION

This mask has a non-transferable three month limited warranty for manufacturer's defects from the date
of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will
replace the mask or its components. For additional information about the Sleepnet Warranty, please visit
www.sleepnetmasks.com. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU
Member State competent authority.

NOTE: The mask has a service life of 6 months.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild
shampoo or dish detergent. Rinse thoroughly after washing. Visually verify that the swivel elbow is not
obstructed and allow the mask to air dry.

NOTE: Inspect mask and valve prior to each use. Replace the mask if parts are damaged or gel is
exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask
under normal use. Follow recommended cleaning and maintenance protocols. The gel cushion is
not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a
dishwashing machine. The warranty does not cover these types of product failures. Therefore, please
handle your new **Mojo**[®] Non Vented Full Face Mask with care.

TECHNICAL INFORMATION

DEADSPACE VOLUME (mL approx.) **Small** 165 mL **Medium** 190 mL **Large** 210 mL
Extra Large 215 mL

Resistance to air flow at 50 L/min: 0.04 cm H₂O

Resistance to air flow at 100 L/min: 0.18 cm H₂O

Operating Temperature: Do not expose the mask to temperatures above 122°F (50°C).

Therapy Pressure: Minimum 3 cm H₂O

USER INSTRUCTIONS FOR **MOJO**[®] HEADGEAR

1. Unhook the right or left magnetic clip from the mask.
2. Place the mask and headgear assembly on head with blue fabric facing outward. Hook the magnetic
clip in place.
3. Adjust upper and lower headgear straps; unhook the Velcro[®] tabs and pull forward to center mask over
face, and adjust strap tension and length. Affix tabs.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Straps
should be adjusted to stabilize mask; pulling the straps too tight can cause skin irritation. The mask
should not constrict your nose or mouth; you should be able to breathe freely and comfortably. If the
mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

4. Attach swivel adapter to device tubing (bi-level, ventilator); turn device on and verify airflow through mask.
5. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit,
and reduce leaks.
6. Rotate the tensioning screw to eliminate leaks at the bridge of the nose. Do not attempt to decrease
bridge leaks by over-tightening the upper strap.

QUICK RELEASE OF MASK AND HEADGEAR

7. Grasp lower strap and lift away from face. Remove mask and headgear assembly.

Mojo[®] is a registered trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com

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Medical Product Services GmbH