

MiniMe[®] 2 Nasal Mask

Quick Start Guide



1 Place the headgear over the patient's head. Position the mask over the nose, ensuring the bottom cushion sits between the upper lip and nose.



2 Thread the bottom two straps through the slots. The Touchless Spacebar should not contact the forehead.



3 Ensure the top headgear strap is fully extended and lies flat across the back of the patient's head, with the headgear logo facing outward.




4 Position the upper and lower straps evenly on both sides of the patient's head. Gradually tighten the headgear straps, starting from the bottom and working your way up. Make sure the upper and lower straps are parallel as shown. The mask should fit comfortably snug, but not too tight. Check for any pressure points or discomfort, and adjust as needed.



5 The MiniMe[®] 2 contains Custom Fit Technology. This soft mask shell can be shaped to follow the contours of a child's face.

Gently press the edges of the mask to change the mask shape, create a comfortable fit, and reduce leaks.

MiniMe[®] 2 Nasal Mask Assembly Guide

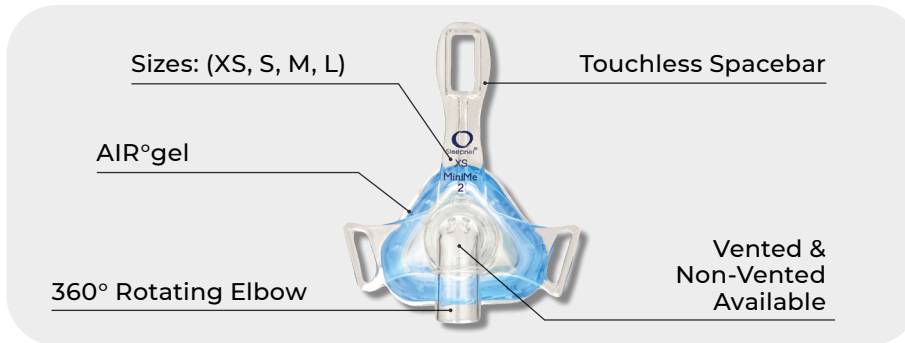

Attach the tubing assembly to the MiniMe[®] 2 mask elbow.



Lay the headgear flat with light gray side facing up with the head strap at the top. Tilt the mask gel side down in the center of the headgear.



Thread the top headgear straps through the top slots of the Touchless spacebar.

NOTES:

- Do NOT overtighten the headgear.
- The Touchless Spacebar should NOT touch the forehead. It should be parallel to the forehead.
- For optimal seal and comfort, the mask should be snug, but not tight.